

# BIJOU THREAD

## BIJOU Thread Guide Book



MADE IN KOREA

- 01 About BIJOU Thread
- 02 Procedure Guide
- 03 Combination of Thread
- 04 Before & After

## Skin Lifting & Tightening

Whitening & Wrinkle Removal

Immediate Effects & Rapid Recovery




Collagen  
Creation  
&  
Fibrotic  
Reaction

Absorbable  
Polydioxanone (PDO)  
thread

High Quality &  
Safety Proved

- Medical Sponge from Germany
- High Quality Needle from Japan
- PDO Thread from Korea



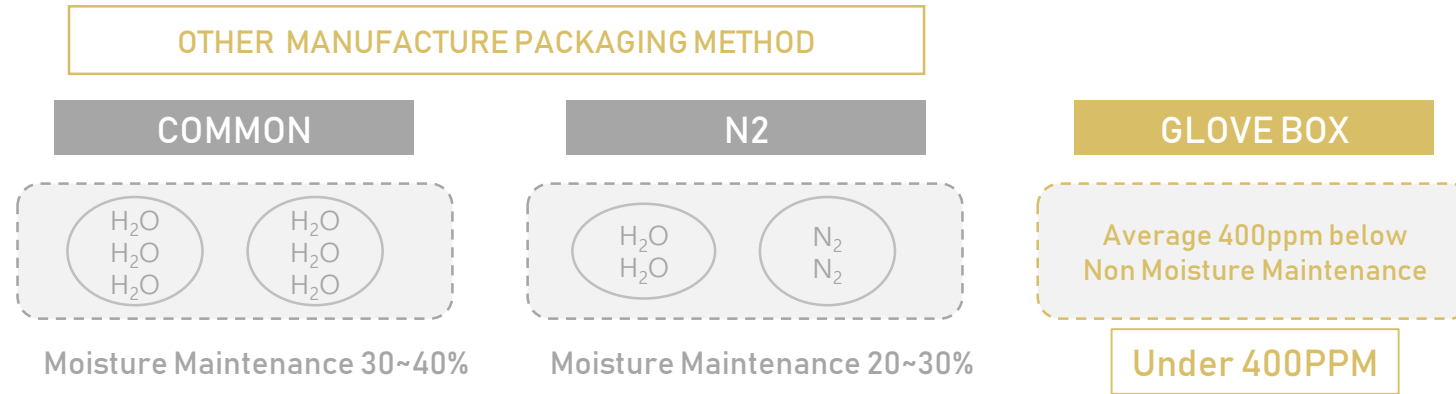


Glove Box

- The Moisture is major factor of low quality. The presence of the moisture affected the decomposition of the polydioxanone.
- Most of manufacturers package N<sub>2</sub> gas or vacuum
- without the remove the moisture in the PDO thread
- However, BIJOU operating by **Glove Box** in order to remove the effects of moisture.



**Result : Not breakable, longer duration**

FIG.9

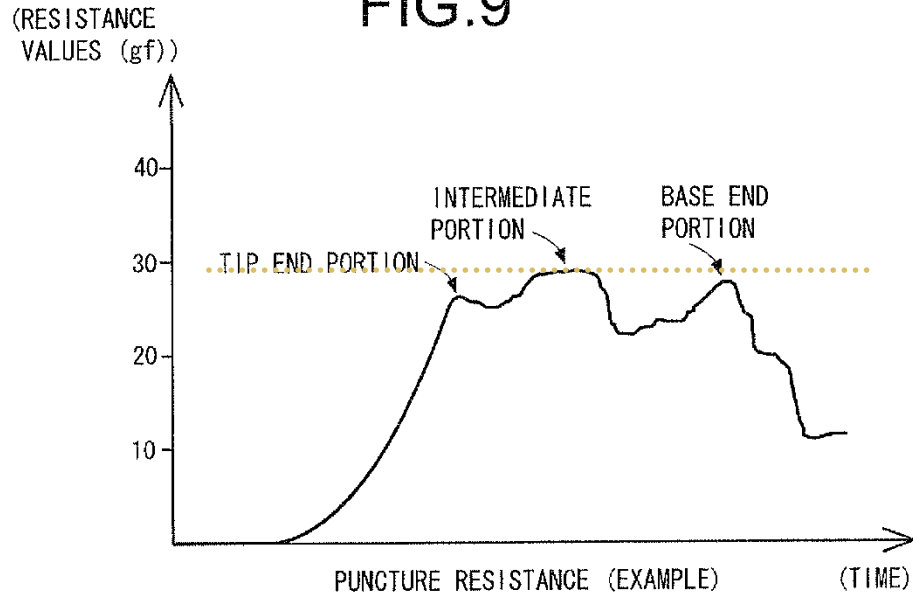


As you see above graph, Resistance value of BIJOU needle is lower than the other companies. Moreover The curve of Graph indicates that resistance against Needle Injection is stable than the other companies.

BIJOU THREAD'S NEEDLE  
FROM JAPAN

Result : Comfortable injection,  
less swelling, bruising, bleeding


UNKNOWN COMPANY

FIG.12

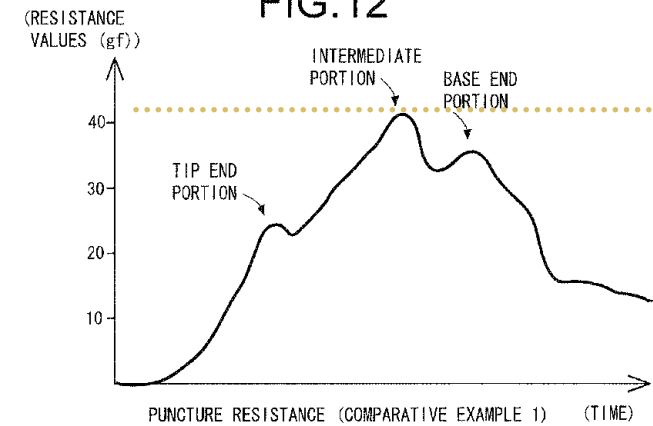

UNKNOWN COMPANY


FIG.13

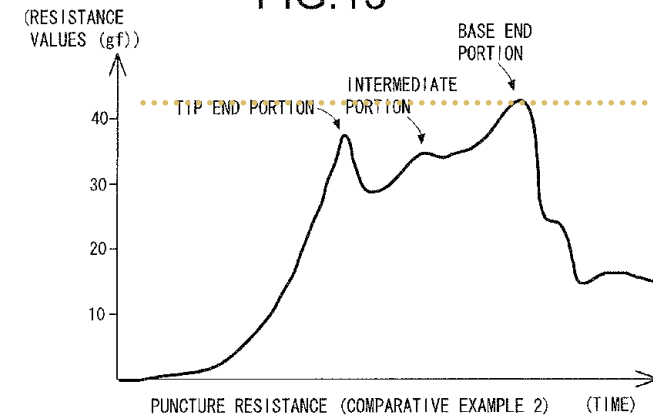

BIJOU Cog thread is absorbable monofilament surgical sutures with cogs (barbed thread) and designed and improved to be maintained longer in the tissue.

Cogs become a supporting structure for soft tissues when the sutures are inserted under the skin.


Thus, such type of Cog thread is much more stable and has long lasting effect.

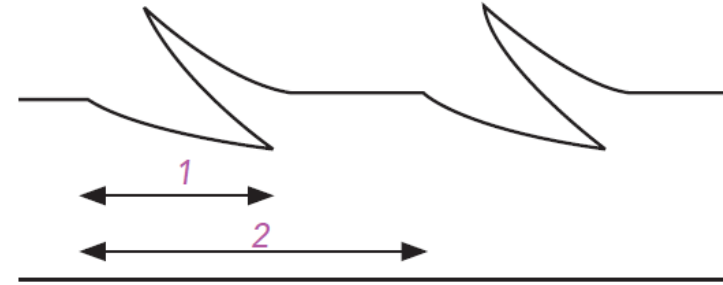

1 is a width of a cog, 2 is a space between cogs.  
The smaller the 2 is and the bigger the 1 is, the stronger the thread is caught though the Production requires more time and effort.




#### Anchoring Strength / Tensile Property

Effective lifting and great clinical uses are due to BIJOU cog thread has greater anchoring and tensile strength than the normal cut- thread offer, has effective facial and body lifting and does not get cut during the insertion procedure.

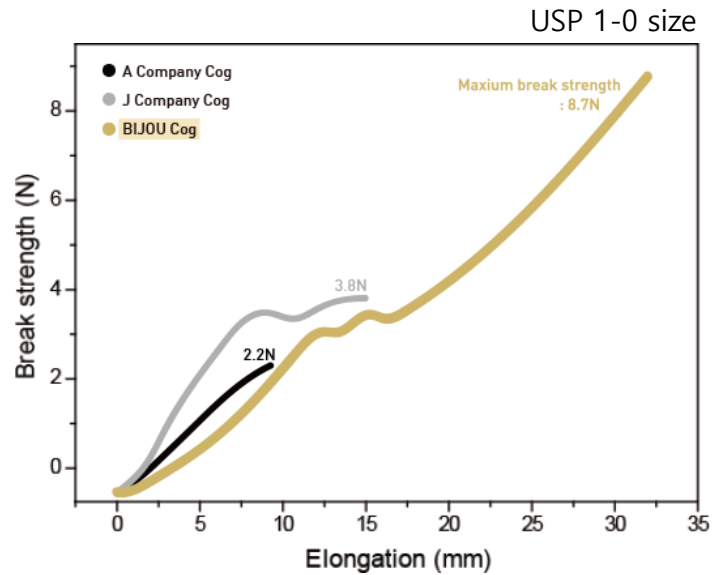

The result of Tensile Force

BIJOU	J COMPANY	A COMPANY
8.7N	3.8N	2.2N

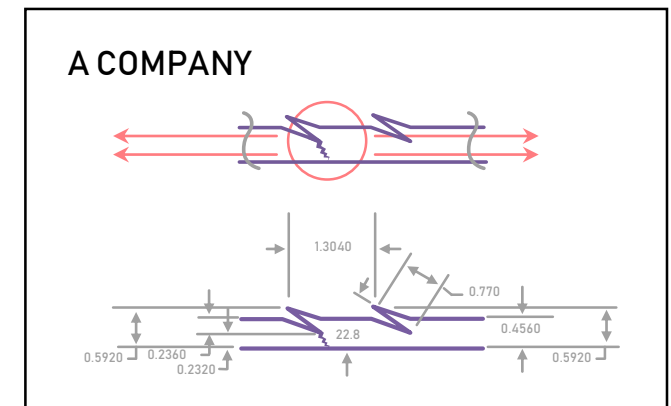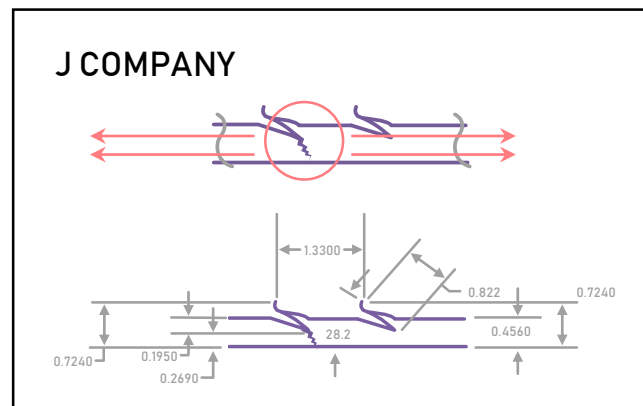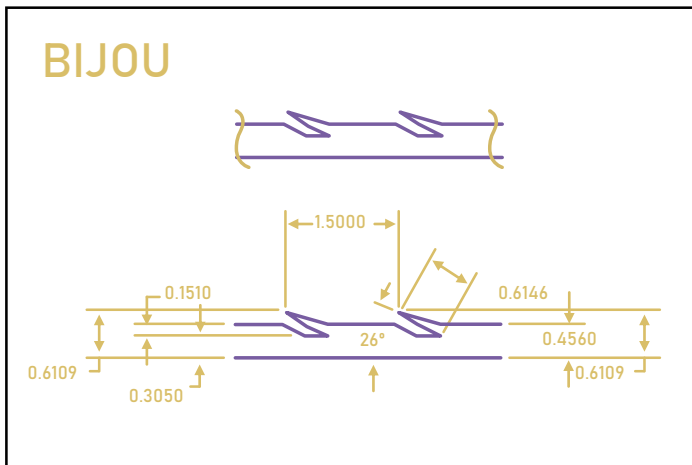

Due to high-technology of Cog's shape and pattern, BIJOU Thread has more powerful tensile and anchoring strength than any other threads.




Choices of the thread or combination of the types of thread depends on the patient's face shape, size of the bone, ages and skin type.




Vertical contraction increases adhesion between threads and derma, muscles or skeleton, and results passive lifting effect, causing sagging skin to cling.

When applying near to solid tissues like bone and ligament, adhesion of skin and fixed tissues is strengthened and fixing effect is increased.

1. Treatment depth
  - Subcutaneous Fat Layer / Fascia / Panniculus Carnosus / Ligament
2. Reasons for not applying in dermis
  - To avoid possible inflammation by irritating sebaceous glands.
  - PDO may be visible on the skin.
  - PDO may come out by movement.

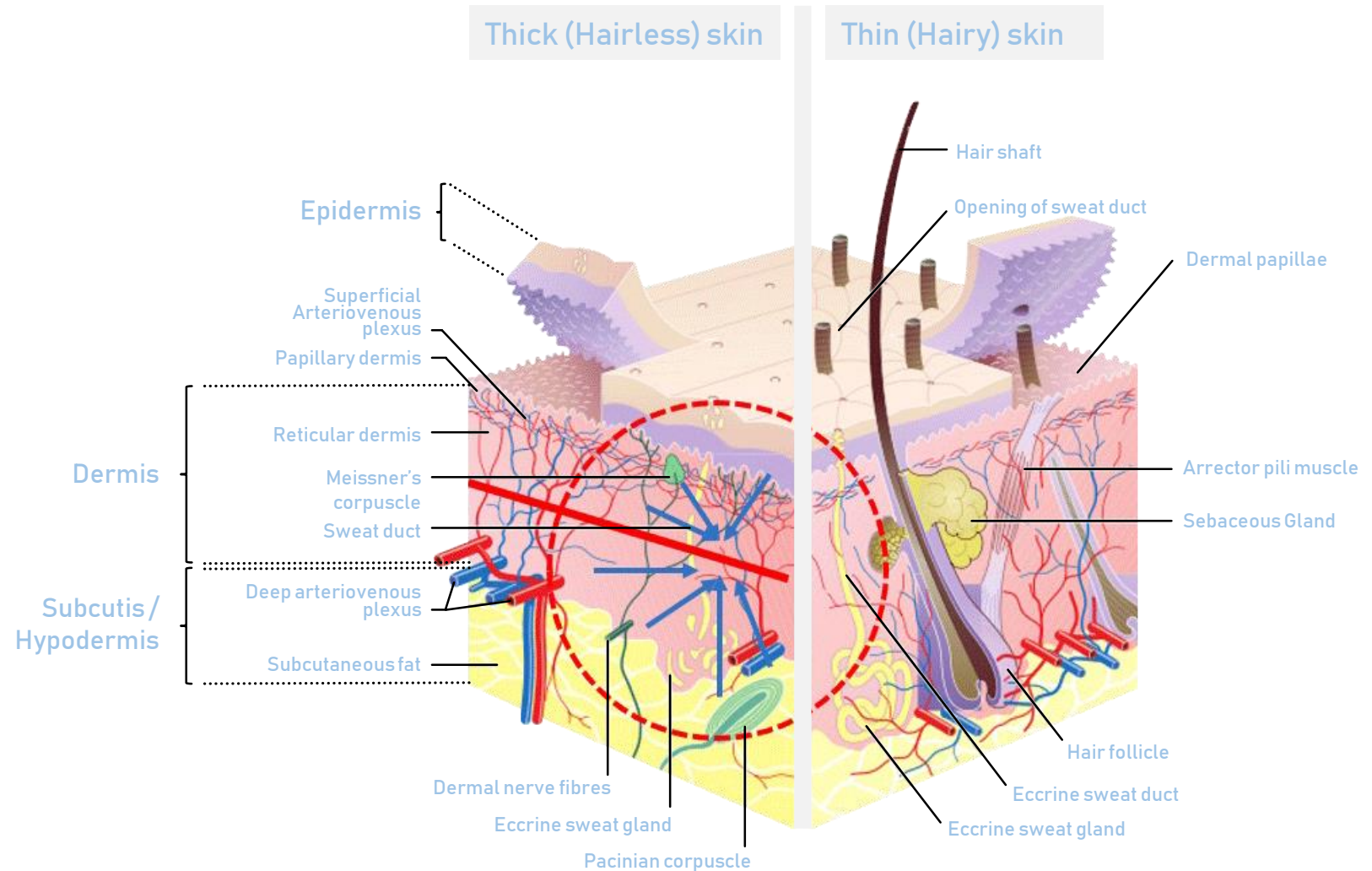

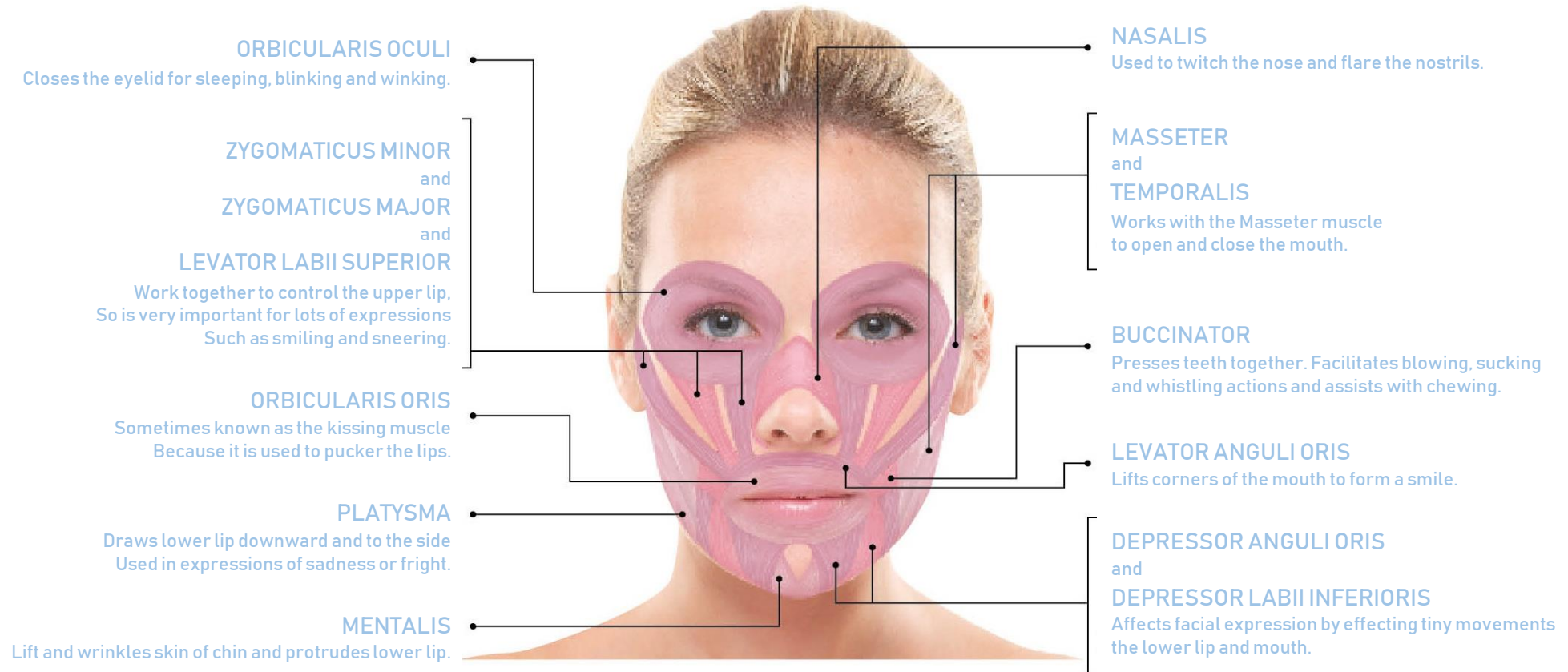

1. Effects on Fat Layer under Skin  
**MONO, DOUBLE, SCREW, DOUBLE SCREW**
2. Effects on Fat Layer under Skin  
**MONO, MONO EYE CANNULA, SCREW, DOUBLE SCREW**
3. Effects on Fat Layer under Skin  
**SCREW, DOUBLE SCREW, DUPLEX, COG EASY 4-0**



MONO / DOUBLE / DUPLEX/ SCREW/  
DOUBLE SCREW / GOLD

- Recommended threads can be different by patient's condition and doctor's discretion



### Effects on Fat Layer under Skin

Obtains directionality as well as contraction effect from the starting point to reference point area.

- Recommended threads can be different by patient's condition and doctor's discretion



### Effects on Fat Layer under Skin

As thread is inserted, stimulus is made and it causes skin contraction and lifting effects.

Volume reduction occurring in jowl liftings also supports the principle of skin contraction effects through stimulus.

- Recommended threads can be different by patient's condition and doctor's discretion



## MONO / DOUBLE / DUPLEX

Do not use snap. Gently press when inserting. (To know the exact depth)  
Be cautious of depth and direction because recipients may have heavy pain or bleeding if threads are applied under the cheekbone.

When the procedure is done too deeply, it damages the blood vessels and may swell up as if wisdom teeth was removed.  
Applying few more thread to chin is recommendable if recipient has developed chin.

- Recommended threads can be different by patient's condition and doctor's discretion






## MONO / DOUBLE / DUPLEX

Do not use snap. Gently press when inserting. (To know the exact depth)  
Be cautious of depth and direction because recipients may have heavy pain or bleeding if threads are applied under the cheekbone.

When the procedure is done too deeply, it damages the blood vessels and may swell up as if wisdom teeth was removed.

Applying few more thread to chin is recommendable if recipient has developed chin.

- Recommended threads can be different by patient's condition and doctor's discretion

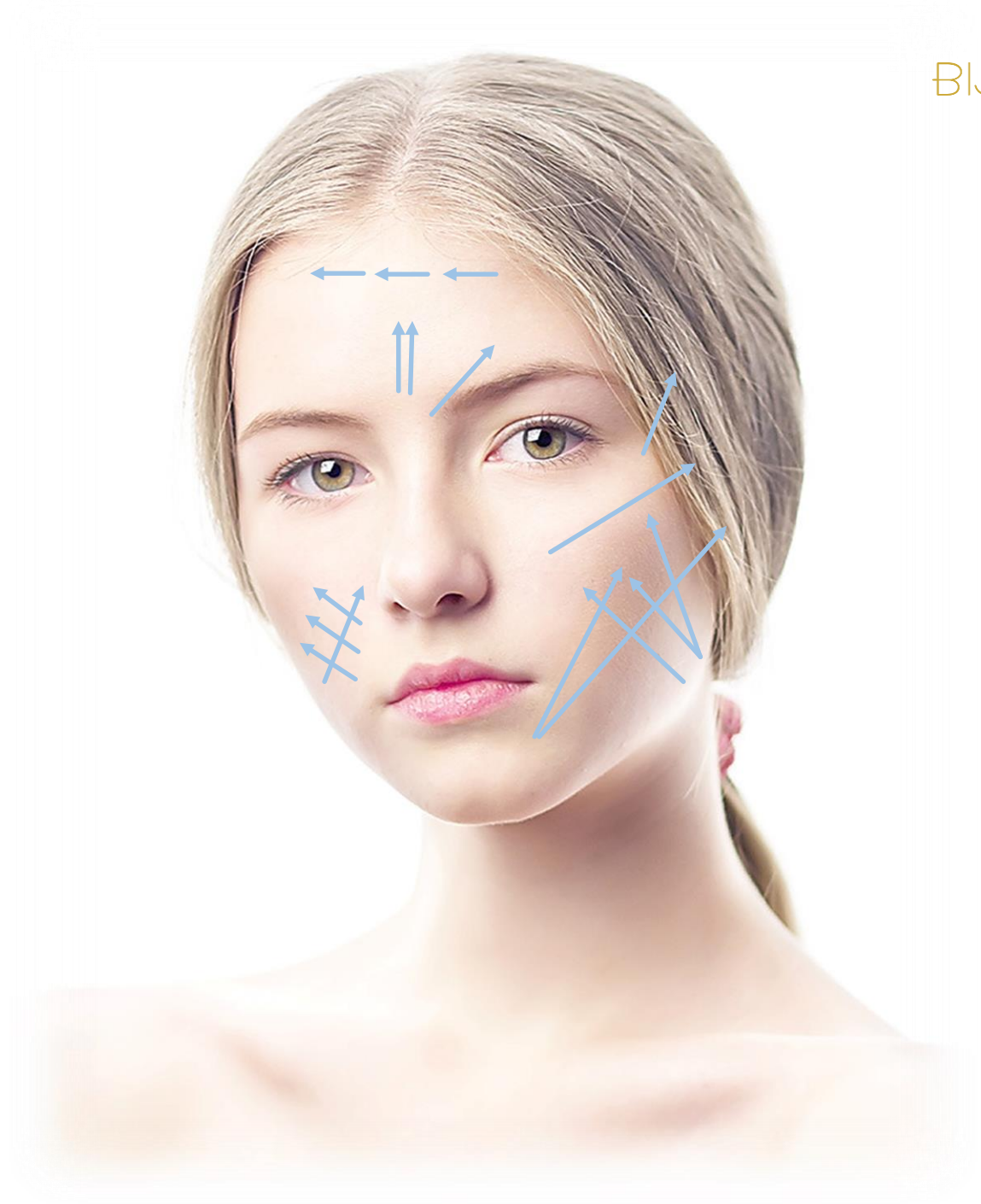

## Anchoring COG Easy 4-0

1. C2338 23G / 38mm / Sharp needle
2. C2360 23G / 60mm / Sharp needle
3. C2390 23G / 90mm / Sharp needle



Effect : Generating Collagen, Volumizing, Lifting

Choose the model considering the size of patient's face and condition

Single-Direction



- Recommended threads can be different by patient's condition and doctor's discretion



**Anchoring COG Easy 4-0**

1. C2338 23G / 38mm / Sharp needle
2. C2360 23G / 60mm / Sharp needle
3. C2390 23G / 90mm / Sharp needle



Effect : Generating Collagen, Volumizing, Lifting

Choose the model considering the size of patient's face and condition




Single-Direction



- Recommended threads can be different by patient's condition and doctor's discretion



## Anchoring COG Basic

1. M19B1014 19G / 100mm / Blunt needle / Bi-Direction   
L19C1015 19G / 100mm / L Cannula / Bi-Direction   
M19T1019 19G / 100mm / Blunt needle / 6D 360 degree cog  
L19T1019 19G / 100mm / L Cannula / 6D 360 degree cog
2. Combination of 21G, 23G Cog Thread
3. Cog Easy 4-0 



Effect : Strong Lifting, Tightening, Collagen Generation  
Wrinkle Improvement

- Recommended threads can be different by patient's condition and doctor's discretion




## Eye (Mono / Twist / Screw)

M3025 : 30G 25mm  
M2938 : 29G 38mm  
M2938C : 29G 38mm  
H2938 : 29G 38mm  
TH2638 : 26G 38mm  
TH2538 : 25G 38mm

Stronger effect

- Recommended threads can be different by patient's condition and doctor's discretion

Effect : Fine wrinkles improvement  
around eyes Generate Collagen, Volumizing

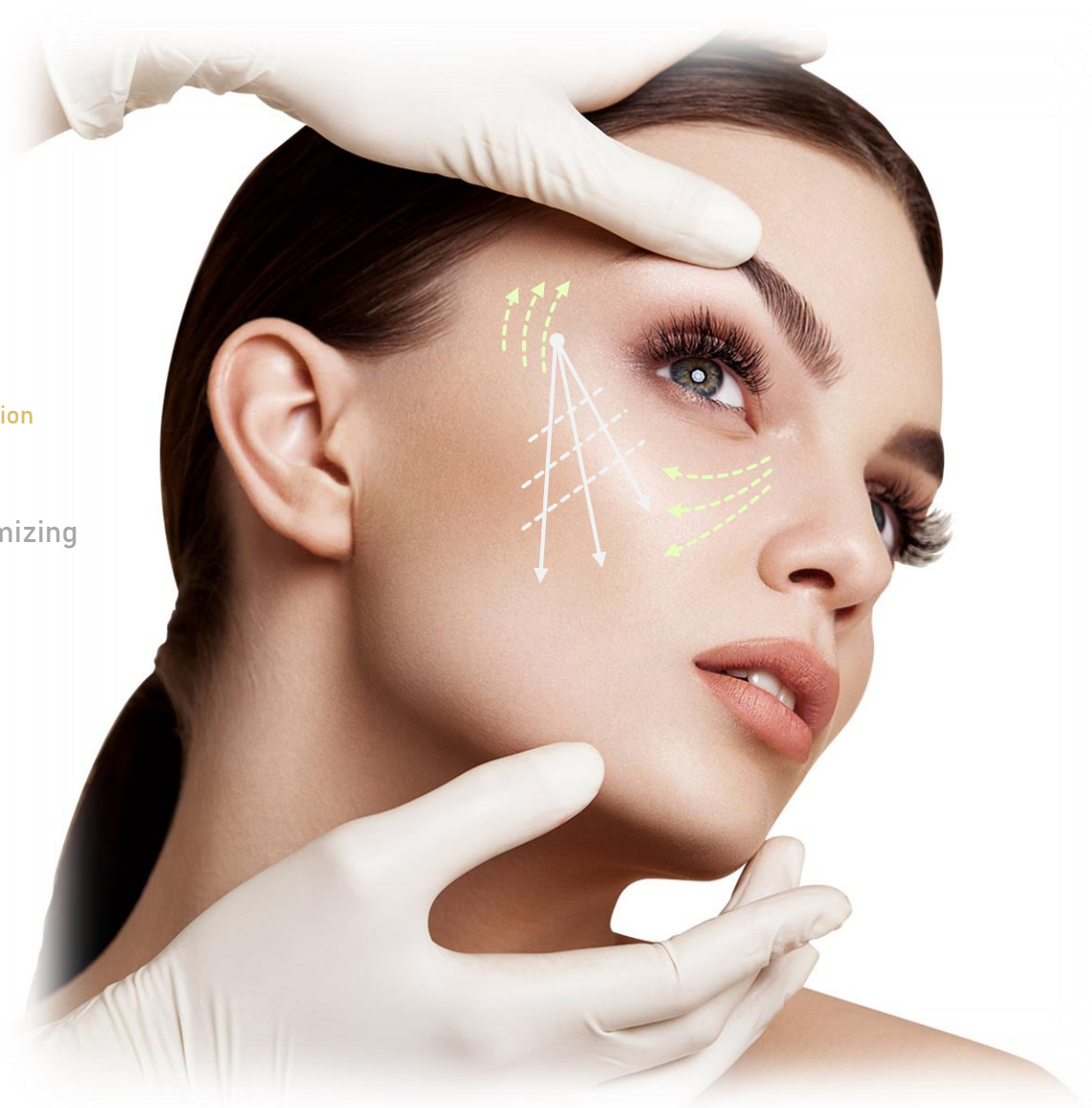

### Eyes (Mono Eye Cannula)

MB3025 : 30G 25mm / B type Cannula



Effect : Fine wrinkles improvement  
around eyes, less bleeding,  
less bruising by **using B type Cannula**

- Recommended threads can be different by patient's condition and doctor's discretion




## Nose (COG CANNULA)

L19E0206 : 19G / 60mm / L-Type Cannula / Bi-Direction L19E0207 : 19G / 70mm / L-Type Cannula / Bi-Direction 

- Recommended threads can be different by patient's condition and doctor's discretion



Effect : Re - Shape nose

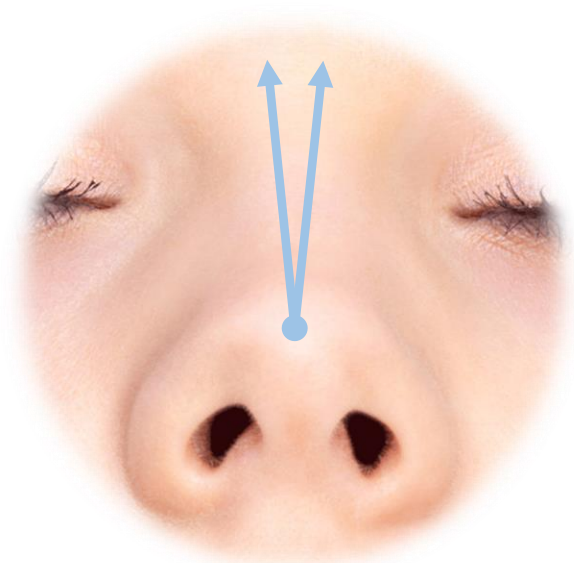



## Nose (Mono / Twist / Screw)

M3025 : 30G 25mm  
M2938 : 29G 38mm  
M2938C : 29G 38mm  
H2938 : 29G 38mm  
TH2638 : 26G 38mm  
TH2538 : 25G 38mm

Stronger effect

- Recommended threads can be different by patient's condition and doctor's discretion

Effect : Generating Collagen, Volumizing

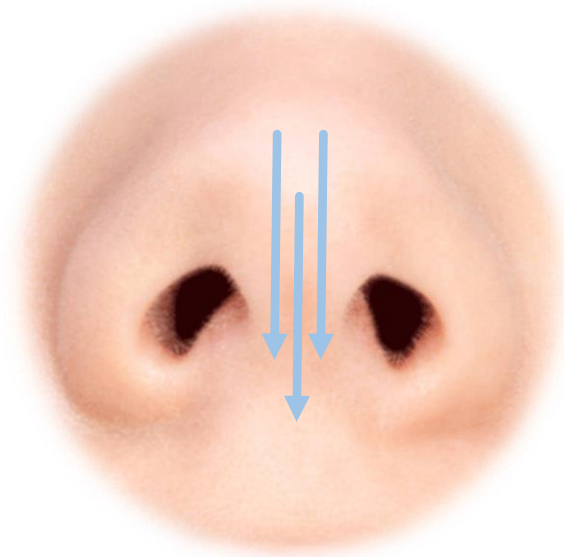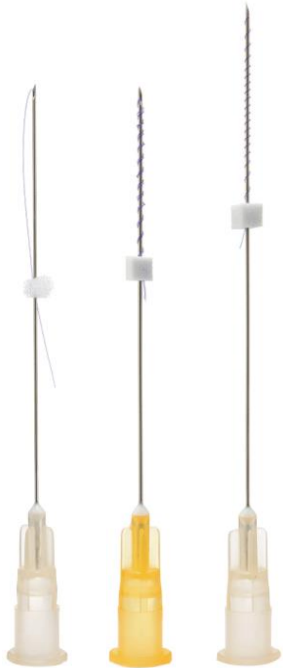




## Nose (Mono / Twist / Screw)

M3025 : 30G 25mm  
M2938 : 29G 38mm  
M2938C : 29G 38mm  
H2938 : 29G 38mm  
TH2638 : 26G 38mm  
TH2538 : 25G 38mm

Stronger effect

- Recommended threads can be different by patient's condition and doctor's discretion

Effect : Generating Collagen, Volumizing

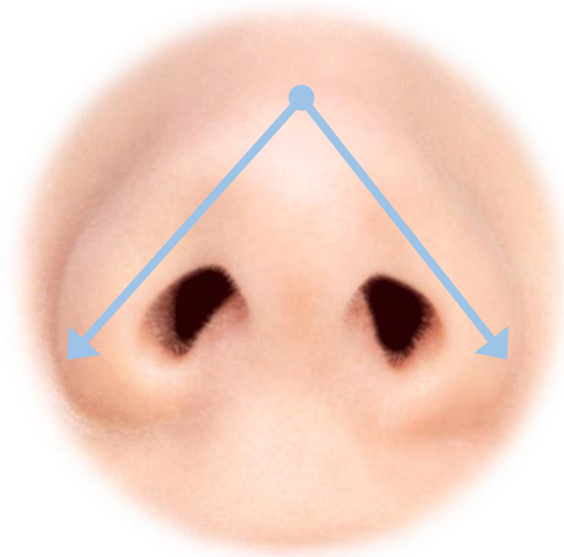

## Abdomen (Mono / Twist / Screw)

M2650 : 26G 50mm  
 M2660 : 26G 60mm  
 M2690 : 26G 90mm  
 T2650 : 26G 50mm  
 T2550 : 25G 50mm  
 T2690 : 26G 90mm  
 H2660 : 26G 60mm  
 H2690 : 26G 90mm  
 TH2690 : 26G 90mm

Stronger effect

Effect : Generating Collagen, Volumizing  
 improvement of wrinkle,  
 Elastic skin, Fat reduce



## Abdomen (Mono / Twist / Screw)

M2650 : 26G 50mm  
 M2660 : 26G 60mm  
 M2690 : 26G 90mm  
 T2650 : 26G 50mm  
 T2550 : 25G 50mm  
 T2690 : 26G 90mm  
 H2660 : 26G 60mm  
 H2690 : 26G 90mm  
 TH2690 : 26G 90mm

Stronger effect

Effect : Generating Collagen, Volumizing  
 improvement of wrinkle,  
 Elastic skin, Fat reduce




- Recommended threads can be different by patient's condition and doctor's discretion

## Buttocks (Mono / Twist / Screw)

M2650 : 26G 50mm  
 M2660 : 26G 60mm  
 M2690 : 26G 90mm  
 T2650 : 26G 50mm  
 T2550 : 25G 50mm  
 T2690 : 26G 90mm  
 H2660 : 26G 60mm  
 H2690 : 26G 90mm  
 TH2690 : 26G 90mm

Stronger effect

Effect : Generating Collagen, Volumizing  
 improvement of wrinkle,  
 Elastic skin, Hip- Up

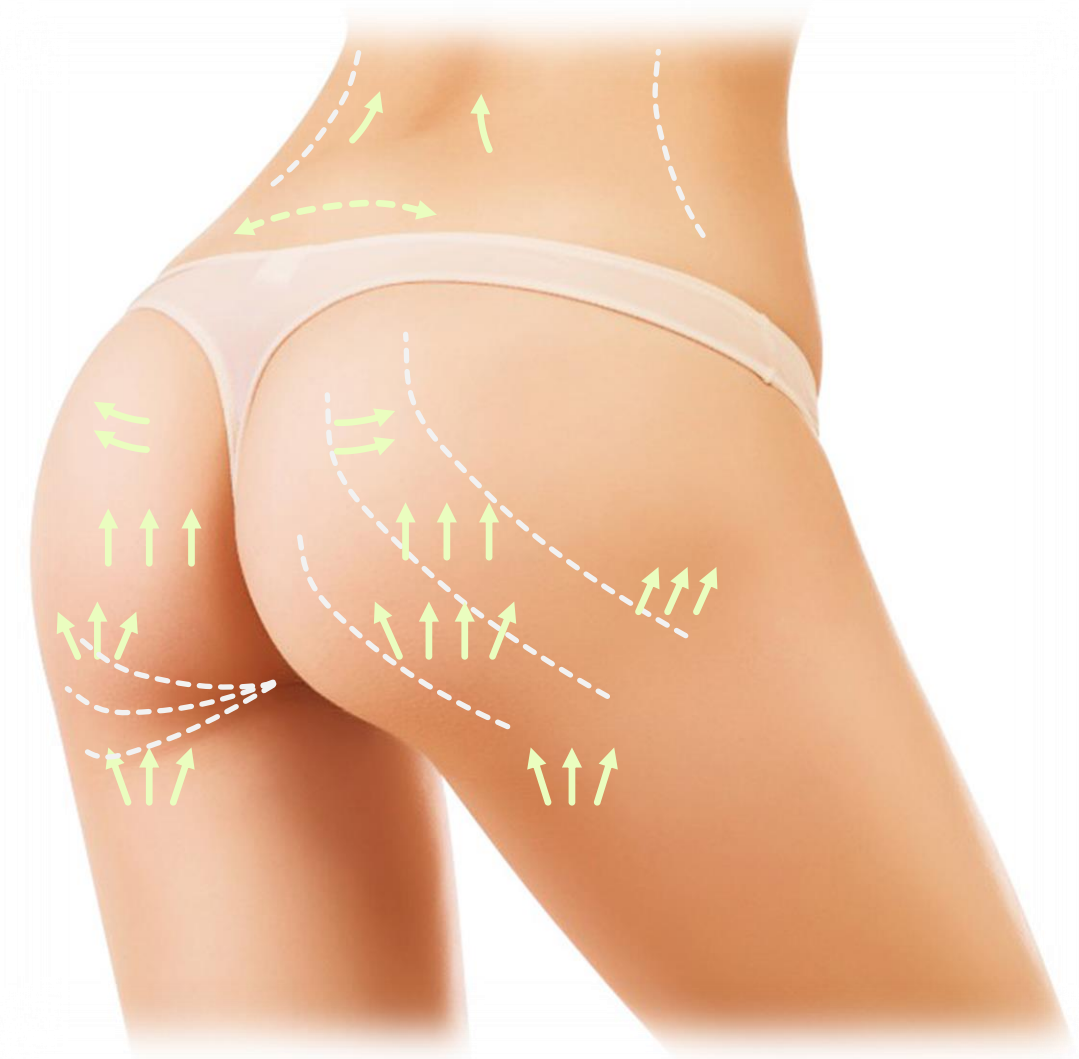

### Buttocks (Mono / Twist / Screw)

M2650 : 26G 50mm  
 M2660 : 26G 60mm  
 M2690 : 26G 90mm  
 T2650 : 26G 50mm  
 T2550 : 25G 50mm  
 T2690 : 26G 90mm  
 H2660 : 26G 60mm  
 H2690 : 26G 90mm  
 TH2690 : 26G 90mm

Stronger effect

Effect : Generating Collagen, Volumizing  
 improvement of wrinkle,  
 Elastic skin, Hip- Up



## Simple Combination



**COG BASIC**  
TYPE I Blunt

Bi-Direction		Multi-Direction	
Gauge	Length	Gauge	Length
19G	100mm	21G	90mm
21G	90mm	23G	60mm
23G	60mm		90mm

2 Types  
(Bi-Direction & Multi-Direction)



**GOLD & DUPLEX**  
TYPE I Sharp

Gold		Duplex	
Gauge	Length	Gauge	Length
26G	38mm	26G	38mm
	50mm		50mm
29G	38mm		
	50mm		

99.99% Pure Gold

PDO + PGA  
(Polyglycolic Acid)



**COG 360°**  
TYPE I Blunt / L-type Cannula

Gauge	Length
19G	100mm
21G	90mm
23G	60mm
	90mm

6 Dimensional 360° Cog



**SCREW**  
TYPE I Sharp


Gauge	Length
26G	60mm
	90mm
29G	38mm
	50mm



**COG CANNULA**  
TYPE I L-type Cannula

Bi-Direction		Multi-Direction		Bi-Direction (Nose Lifting)	
Gauge	Length	Gauge	Length	Gauge	Length
19G	100mm	21G	90mm	19G	60mm
21G	90mm				70mm

2 Types  
(Bi-Direction & Multi-Direction)

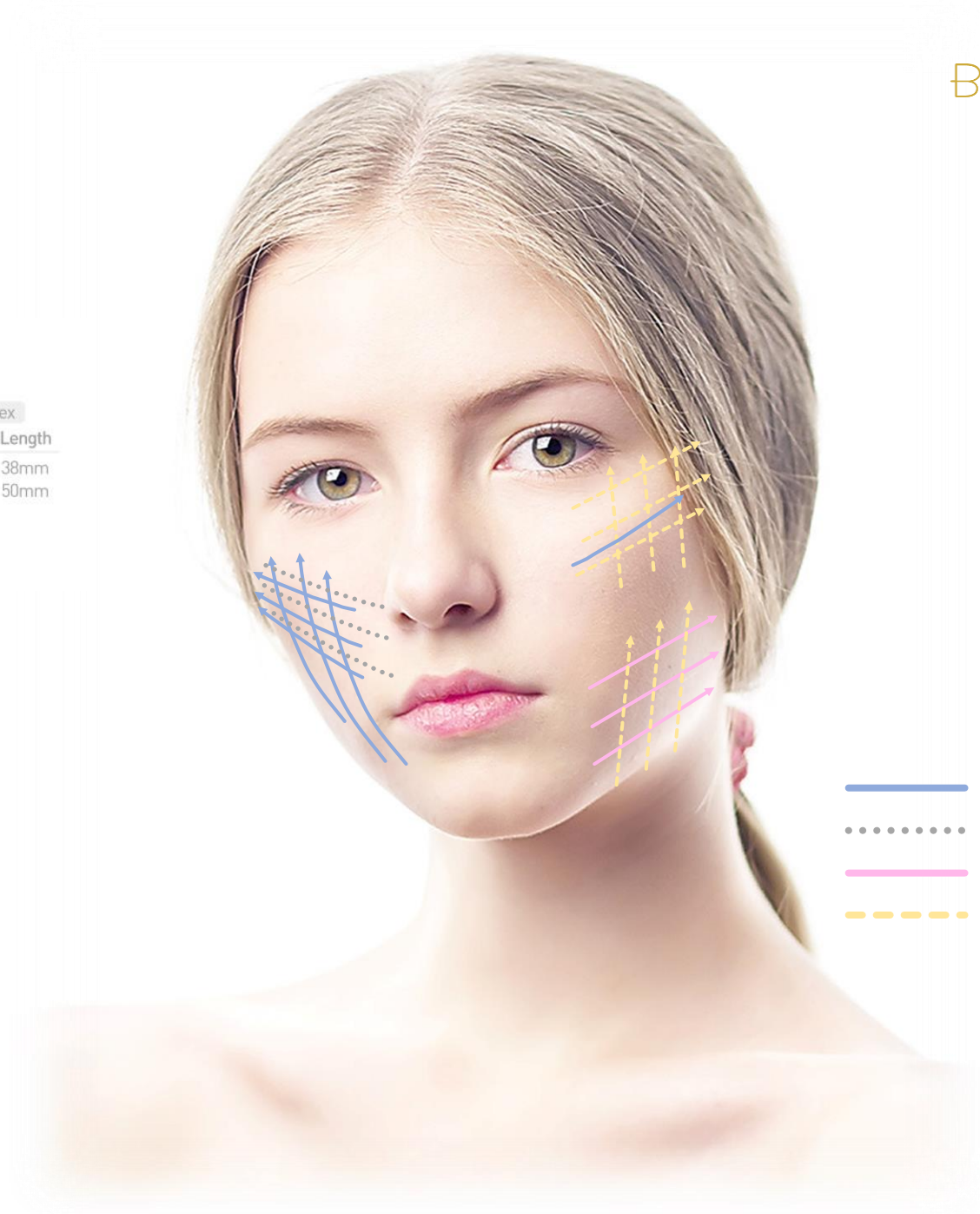

- Cog
- Duplex
- Screw
- Gold

- Recommended threads can be different by patient's condition and doctor's discretion
- Please choose the model considering patient's size of face and condition

## Common Combination

MONO & MONO WHITE  
TYPE I Sharp

Mono		Mono White	
Gauge	Length	Gauge	Length
26G	38mm	26G	38mm
	50mm		50mm
	60mm		60mm
29G	90mm	29G	90mm
	38mm		38mm
	50mm		50mm
30G	25mm		38mm
	38mm		

MONO EYE CANNULA  
TYPE I B-type Cannula


Gauge	Length
30G	25mm

GOLD & DUPLEX  
TYPE I Sharp

Gold		Duplex	
Gauge	Length	Gauge	Length
26G	38mm	26G	38mm
	50mm		50mm
29G	38mm		50mm
	50mm		

DOUBLE & DOUBLE SCREW  
TYPE I Sharp

Gauge	Length
25G	38mm
	50mm
26G	38mm
	50mm
	90mm

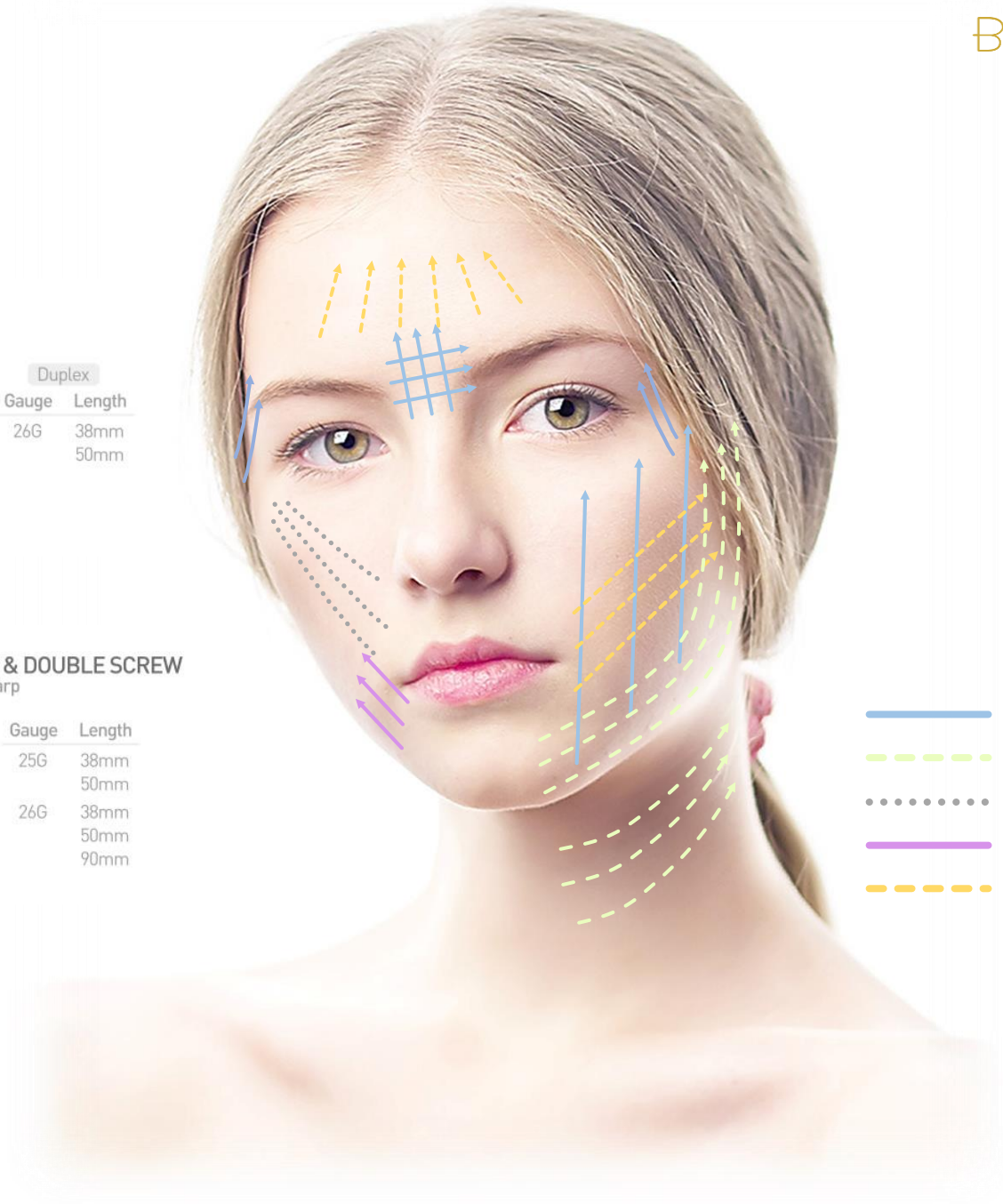

- Mono
- - - Double
- ..... Duplex
- Screw
- - - Gold

- Recommended threads can be different by patient's condition and doctor's discretion
- Please choose the model considering patient's size of face and condition

## Full Combination

**MONO & MONO WHITE**  
TYPE I Sharp

Mono		Mono White	
Gauge	Length	Gauge	Length
26G	38mm	26G	38mm
	50mm		50mm
	60mm		60mm
29G	90mm	29G	90mm
	38mm		38mm
	50mm		50mm
30G	25mm		38mm

Transparent

**GOLD & DUPLEX**  
TYPE I Sharp

99.99% Pure Gold

PDO + PGA (Polyglycolic Acid)

Gold		Duplex	
Gauge	Length	Gauge	Length
26G	38mm	26G	38mm
	50mm		50mm
29G	38mm		
	50mm		

**DOUBLE & DOUBLE SCREW**  
TYPE I Sharp

Gauge	Length
25G	38mm
	50mm
26G	38mm
	50mm
	90mm

**COG CANNULA**  
TYPE I L-type Cannula

2 Types (Bi-Direction & Multi-Direction)

Bi-Direction		Multi-Direction		Bi-Direction (Nose Lifting)	
Gauge	Length	Gauge	Length	Gauge	Length
19G	100mm	21G	90mm	19G	60mm
21G	90mm				70mm

**COG BASIC**  
TYPE I Blunt

2 Types (Bi-Direction & Multi-Direction)

Bi-Direction		Multi-Direction	
Gauge	Length	Gauge	Length
19G	100mm	21G	90mm
21G	90mm	23G	60mm
23G	60mm		90mm
	90mm		


**COG 360°**  
TYPE I Blunt / L-type Cannula

6 Dimensional 360° Cog

Gauge	Length
19G	100mm
21G	90mm
23G	60mm
	90mm

**SCREW**  
TYPE I Sharp

Gauge	Length
26G	60mm
	90mm
29G	38mm
	50mm

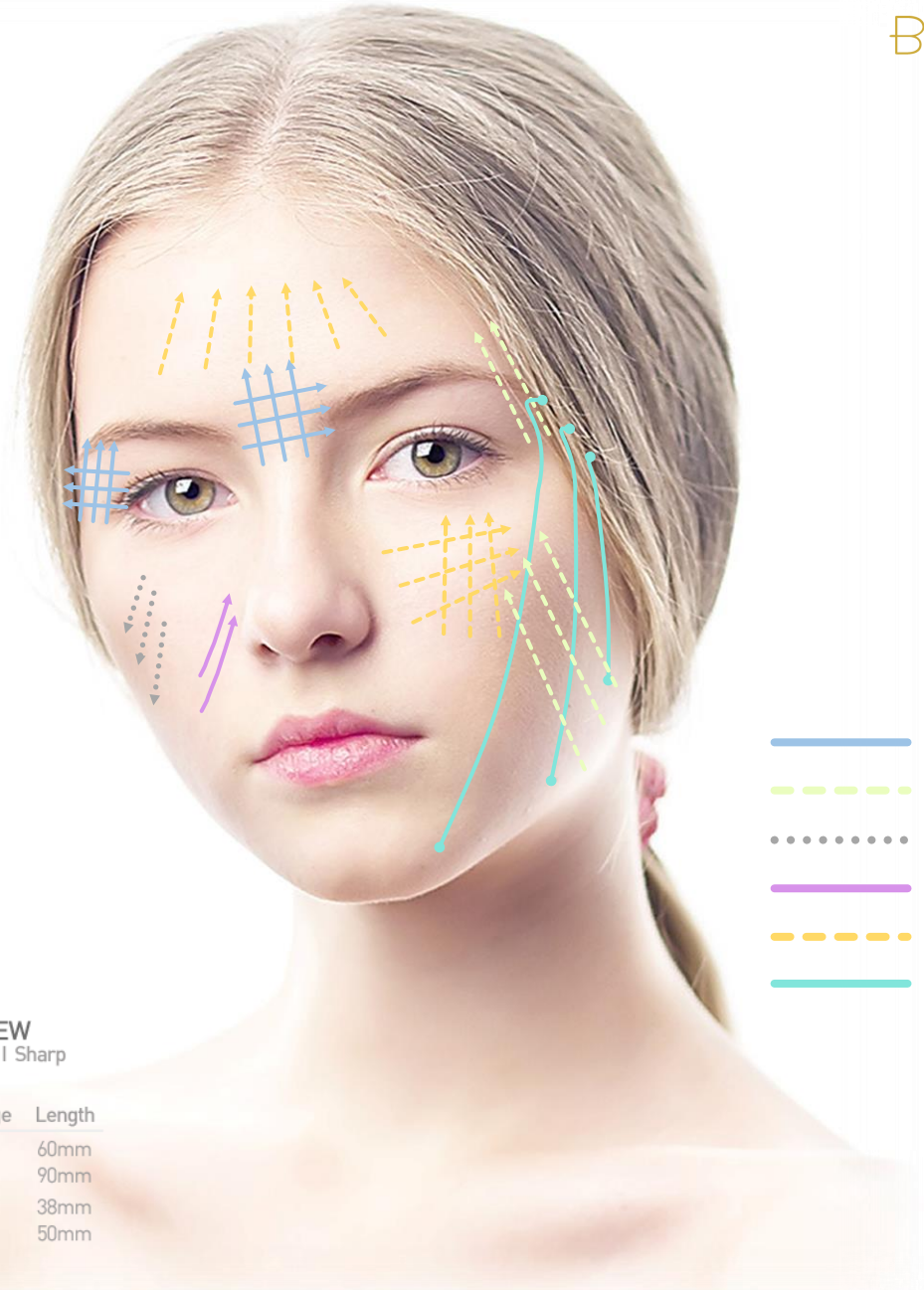

- Mono
- Double
- Duplex
- Screw
- Gold
- Cog



## Combination of Botox, Filler and Threads

**MONO & MONO WHITE**  
TYPE I Sharp

Mono		Mono White	
Gauge	Length	Gauge	Length
26G	38mm	26G	38mm
	50mm		50mm
	60mm		60mm
29G	90mm	29G	90mm
	38mm		38mm
	50mm		50mm
30G	25mm		38mm
	38mm		

Transparent

**GOLD & DUPLEX**  
TYPE I Sharp

Gold		Duplex	
Gauge	Length	Gauge	Length
26G	38mm	26G	38mm
	50mm		50mm
29G	38mm		50mm
	50mm		

99.99% Pure Gold  
PDO + PGA (Polyglycolic Acid)

**DOUBLE & DOUBLE SCREW**  
TYPE I Sharp

Gauge	Length
25G	38mm
	50mm
26G	38mm
	50mm
	90mm

**COG CANNULA**  
TYPE I L-type Cannula

Bi-Direction		Multi-Direction		Bi-Direction (Nose Lifting)	
Gauge	Length	Gauge	Length	Gauge	Length
19G	100mm	21G	90mm	19G	60mm
21G	90mm				70mm

2 Types (Bi-Direction & Multi-Direction)

**COG BASIC**  
TYPE I Blunt


Bi-Direction		Multi-Direction	
Gauge	Length	Gauge	Length
19G	100mm	21G	90mm
21G	90mm	23G	60mm
23G	60mm		90mm
	90mm		

2 Types (Bi-Direction & Multi-Direction)

**COG 360°**  
TYPE I Blunt / L-type Cannula

Gauge	Length
19G	100mm
21G	90mm
23G	60mm
	90mm

6 Dimensional 360° Cog

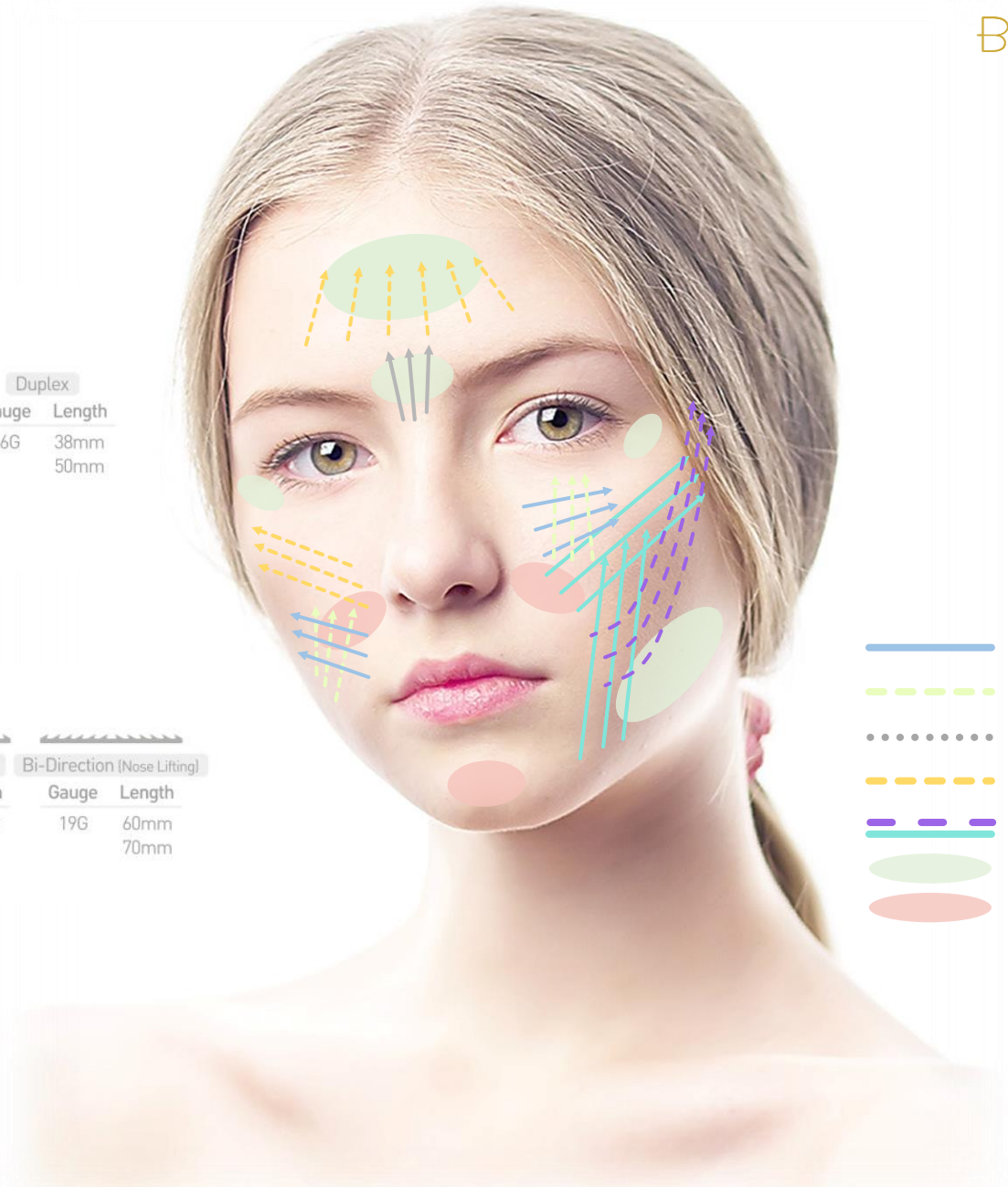

- Mono
- Double
- Duplex
- Gold
- Cog
- Botox
- Filler



